


Directions for Use - PreStained Morphology Slides


STEP 1

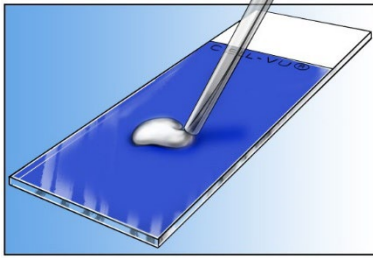

Apply a small drop (approximately 2-4 microliters) of specimen on the center of the slide.


NOTE: EDTA is the recommended anticoagulant for venous blood. Flouride oxalate and heparinized blood are NOT suitable.

STEP 2

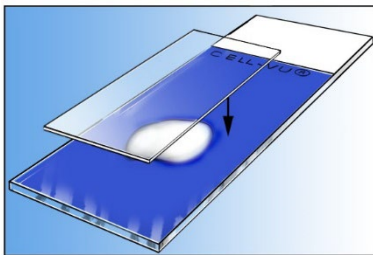

Place the coverglass (24 x 24 mm or 24 x 40 mm) on top of the slide, exerting slight pressure on the coverglass.

STEP 3

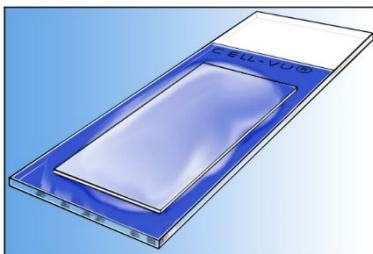

Allow the slide to stand at room temperature for 15 minutes before microscopic observation is made. The slides are used for evaluating sperm head/acrosome size, differentiation of white blood cells in peripheral blood and cytology of body fluids.